

PRODUCT CATALOGUE

PRODUCT : **SURGICAL RUBBER GLOVES --STERILE-POWDER FREE - ORTHOPEDIC**
BRAND NAME : **AGE.CARE**
COMPLYING STANDARDS : **ASTM D 3577:2009/ EN 455-1:2000, EN 455-2:2011 and EN 455-3:2006**
Average weight for size 7½ : 24 gm/Pair

Specification

- Made from high quality natural rubber latex
- Colour : Natural white
- Anatomically shaped curved fingers; hand specific
- Micro rough surface texture provides enhanced instrument grip
- Substantially low protein level to minimize the risk of allergen
- Ring rolled bead and extra length for extra protection
- Quality standard as per AQL1.5 or better.
- Sterilized by ETO



Physical Dimensions

SIZE	6.0"	6½"	7.0"	7½"	8.0"	8½"	9.0"	Tol.
Length (mm)	280							Min
Width (Palm) mm	76	83	89	95	102	108	114	±6
Thickness at (mm)	0.21	Palm		0.23	Finger		0.25	

Physical Properties

Sr. No.	Parameter	Before Ageing	After Ageing 70±2°C for 166±2 h
01	Tensile Strength MPa Min	24	18
02	Elongation at Break (%) Min	750	560
03	Stress at 500% Elongation (Mpa) Max	5.5	NA
04	Powder content	<2mg/dm ²	

Expiry Date

The shelf life of the gloves is 3 years from the date of manufacture.

USER INSTRUCTIONS

MEDICAL DEVICE: SURGICAL RUBBER GLOVES --STERILE-POWDER FREE - ORTHOPEDIC

Intended applications:

Surgical Rubber Gloves –These gloves are intended for use in the healthcare sector to provide extra protection during rigorous and complicated orthopedic surgical procedures particularly bone fracture cases.

Contra indications:

Latex Surgical gloves are manufactured from NATURAL RUBBER LATEX

CAUTION!

“THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTION TO SOME INDIVIDUALS. IF ANY ALLERGIC REACTION IS OBSERVED DISCONTINUE THE USE AND CONSULT A PHYSICIAN”

Instructions for use

- Each pouch contains one pair separate gloves for left and right hand.
- Size No is marked for Selection of gloves of appropriate size
- Must see the date of Manufacturing and expiry date , Lot/Batch No & Manufacturers address
- Use micro roughened glove for added instrument grip
- Do not use the sterilized gloves if package is opened or damaged. Do not re-sterilize damaged or opened pouches
- The device is for single use only
- Limit “Touch Contamination” with soiled gloves
- Do not store gloves exposed to direct sunlight, fluorescent lighting or under high Humidity areas and protect from Ozone atmosphere.
- Store gloves under cool dry place where temperature does not rise above 40°C
- Disposal after use as per local, state, national regulatory rules

Packing:

One pair per pouch

50 Pairs per Dispenser cartons

400 Pairs per Master cartons